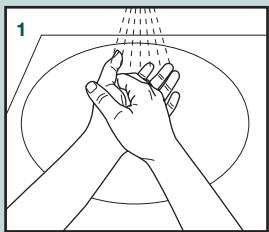
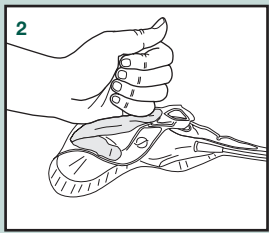
Deutsch

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English

Before use, wash hands thoroughly and disinfect them.

Open the water sachet by folding and squeezing.

Make sure the whole content of the sachet is emptied into the packaging and all of the catheter surface is activated by the fluid for 30 sec.

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Open the urine bag at the front end and release the tip of the catheter. Avoid touching any part of the catheter surface (except for the funnel) directly.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Push back the catheter into the urine bag and seal this by making a knot. To empty the bag tear at the back end.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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